



Advanced Skin and Wound Cleanser Advanced Wound Irrigation Solution

Instructions for Use

INGREDIENTS

Electrolyzed Water, Sodium Chloride (NaCl), and preservatives: Sodium Hypochlorite (NaOCl), and Hypochlorous Acid (HOCl).

Note: The solution contains sodium hypochlorite and hypochlorous acid as preservatives to reduce/prevent the growth of microorganisms within the solution.

DESCRIPTION

Renovar™ Advanced Skin and Wound Cleanser and Renovar™ Advanced Wound Irrigation Solution is a non-irritating, non-cytotoxic, non-sensitizing, non-oily, non-foaming pH neutral formulation indicated for use as described below.

INDICATIONS FOR USE

Renovar Advanced Skin and Wound Cleanser and Renovar Advanced Wound Irrigation Solution are intended for the cleansing, irrigation, moistening, debridement and removal of foreign material and debris from exuding wounds, acute and chronic dermal lesions including stage I-IV pressure ulcers, stasis ulcers, diabetic ulcers, post-surgical wounds, first- and second-degree burns, abrasions, minor irritations of the skin, diabetic foot ulcers, ingrown toenails, grafted/donor sites and exit sites. It is also intended for use to moisten and lubricate wound dressings and for use with devices intended to irrigate wounds.

CONTRAINDICATIONS

Known hypersensitivity to any of the Renovar Advanced Skin and Wound Cleanser and Renovar Advanced Wound Irrigation Solution ingredients.

PRECAUTIONS

Renovar Advanced Skin and Wound Cleanser and Renovar Advanced Wound Irrigation Solution are to be used only as directed by a licensed healthcare practitioner. Do not use to treat any condition other than that for which it is prescribed. Avoid contact with the eyes.

If a reaction to Renovar Advanced Skin and Wound Cleanser and Renovar Advanced Wound Irrigation Solution suggesting sensitivity or chemical irritation occurs, use of the medication should be discontinued and the prescribing healthcare practitioner consulted.

WARNING

For external use only. Not for injection.

SIDE EFFECTS (Possible)

There may be a temporary burning or stinging sensation when first applied to broken skin.

DIRECTIONS FOR USE

Use Renovar Advanced Skin and Wound Cleanser and Renovar Advanced Wound Irrigation Solution to clean and aid debridement of wounds and peri-wound tissue. Do not rinse. To moisten a dressing, apply to wound dressing and cover wound bed. To keep dressing moist, apply Renovar as needed throughout the day.

HANDLING

No special handling precautions required.

STORAGE

Store with cap closed tightly after each use in its original container. Store at 59 to 77°F (15 to 25°C) away from direct sunlight and heat sources. Do not freeze.


DISPOSAL

No special disposal considerations required.

For Topical Use Only

Federal Law restricts this device to sale by or on the order of a licensed healthcare practitioner.

KEEP OUT OF THE REACH OF CHILDREN.

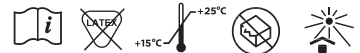

Symbols glossary and additional information available at healwithrenovar.com.

Distributed by:
HR Pharmaceuticals, Inc.
2600 Eastern Blvd., Suite 201
York, PA 17402 USA
877-302-1110
hrpharma.com

Renovar is a part of the **HR** family of brands.

HR and Renovar are trademarks of HR Pharmaceuticals, Inc.
©2022 HR Pharmaceuticals, Inc. All rights reserved.